




Assembly Instructions

Riverside, CA 92506 USA


1-800-800-7871

www.themagicrail.com


1. Push pin on B1 and insert it to A1 & adjust it to the width of the bed for Twin or Full size bed.

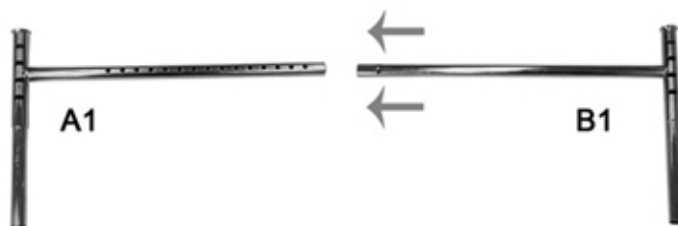


5. Insert the clamps (D) on the 4 posts of set 1 & 2 to safely secure the rail assembly onto the box spring.

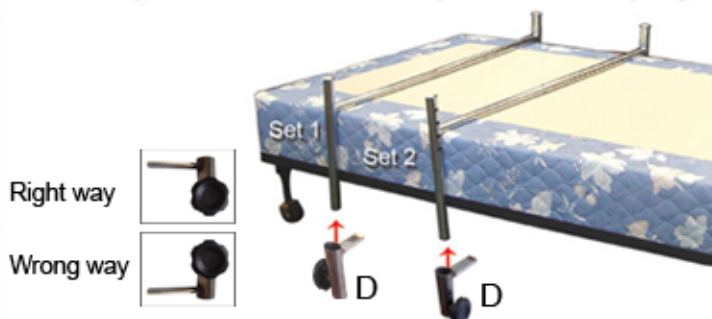


2. If the bed is Queen or King Size insert extension tube between A1 & B1 & adjust it to the width of the bed.

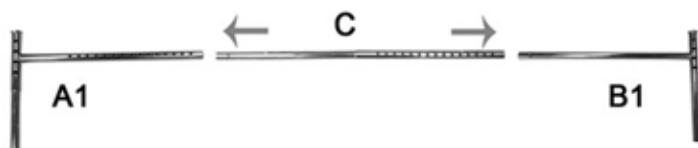

6. For **Single Safety Rail** place the rail (E) where the side rail is intended to be used and part (F) on the other side.

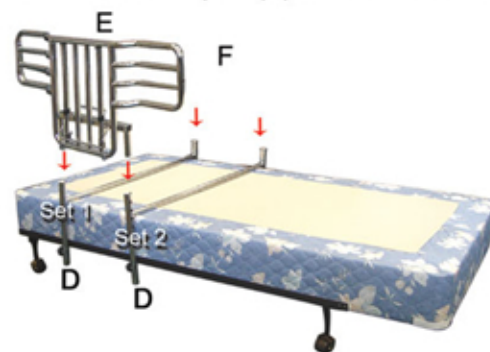


- For **Double Safety Rail** place each rail on each side.

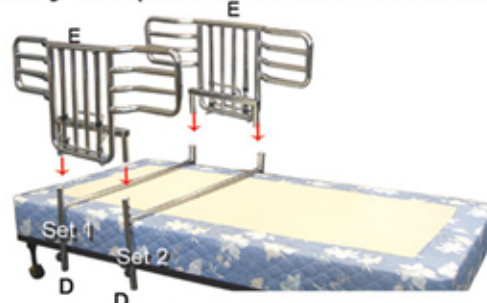


- * Do not forget to adjust the height of the rail to the thickness of the mattress.

3. Repeat procedure 1 or 1&2 depending on the size of the bed, for the other set of parts (A2 & B2 & C). It should look like this:

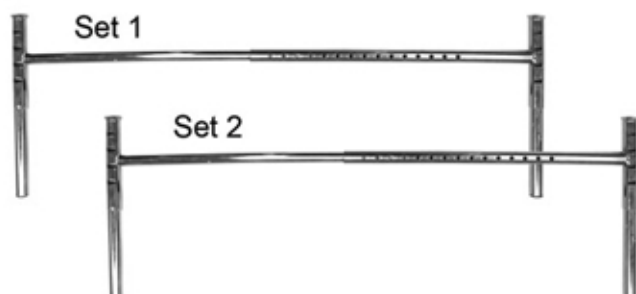

4. Place Set 1 & 2 over the box spring as shown in the picture. **For adjustable electric beds make sure that set 2 must be placed exactly where the bed bents.**

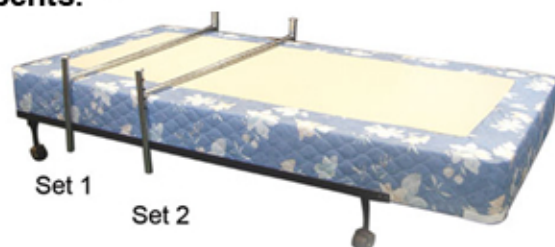

7. Finally, put mattress on top of the box spring with the assembled rail/s.

